

## **PACS Keyboard Shortcuts**

## Access PACS: pacsviewer.searad.com

Keyboard Shortcut		Action
Press and hold (>1 sec)	•	Scroll through the next images
Press and hold (>1 sec)	<b>—</b>	Scroll through the previous images
Į		Next series
	t	Previous series
	Insert	Middle image in series
F	Home	First image in series
	End	Last image in series
	<b>-</b>	Next image in series
	<b>-</b>	Previous image in series

For questions or help with Seattle Radiology PACS please contact, 206.233.7700 or helpdesk@Radiax.com

