

























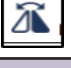


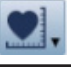


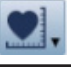






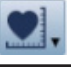






















Icon	Description												
	Study Search - Display the Study Search screen. Note: If launching via an external system, this option may be disabled.												
	Navigator - Access to view a structured report, key image notes or studies in the ZFP viewer.												
	Series Selector - Access to comparison exams for the selected patient.												
	Layout - Change the layout of the active viewport. The toolbar shows the currently displayed layout in the active viewport.												
	Sheet View Layout - Hover over this icon to create a sheet view layout in the active viewport. Once the sheet view is created, the sheet view controls display in the toolbar. 												
	Scroll - Scroll through images in a series.												
	Zoom - Zoom in and out on an image. The dropdown menu includes: <ul style="list-style-type: none"> • Original Resolution: Resize the image to its original captured image size. • Zoom to Fit: Resize the image to its maximum size inside the viewport without cropping. • 2x: Resize the image two times the original resolution. • 3x: Resize the image three times the original resolution. • 4x: Resize the image four times the original resolution. 												
	Pan - Activate the pan tool.												
	Window/level - Activate the window/level tool.												
	Invert - Invert the grayscale of the image in the active viewport.												
	Flip Rotate - Options in the drop-down menu include: <ul style="list-style-type: none">  Rotate an image to the right.  Flip an image vertically.  Flip an image horizontally. 												
	Annotations and Measurements - Options in the dropdown menu include: <table border="1" data-bbox="303 1758 1513 1995"> <tbody> <tr> <td> Distance</td> <td> Ellipse</td> <td> Text</td> <td> CTR Measurement</td> </tr> <tr> <td> Angle</td> <td> Freehand</td> <td> Arrow</td> <td> Rectangular Shutter</td> </tr> <tr> <td> Elliptical Region</td> <td> Rectangle</td> <td></td> <td></td> </tr> </tbody> </table>	 Distance	 Ellipse	 Text	 CTR Measurement	 Angle	 Freehand	 Arrow	 Rectangular Shutter	 Elliptical Region	 Rectangle		
 Distance	 Ellipse	 Text	 CTR Measurement										
 Angle	 Freehand	 Arrow	 Rectangular Shutter										
 Elliptical Region	 Rectangle												

	Overlay - Show or hide the image overlays.
	Cross Reference - Turn on or turn off the cross-reference lines.
	Presentation State - Apply a different presentation state or reset a presentation state.
	Bookmarked Images - Apply a bookmark from a drop-down list of bookmarks referenced in the current study. This icon is disabled when no bookmarks are referenced in a study.
	Synchronize - Synchronize parallel series. This icon is disabled for multi-frame images.
	External Application - Launch a web-based external application from the drop-down menu.
	Share Study URL - Copy or email a study URL. The email recipient can open the study by clicking the URL from the email.
	Reports/Notes - Display a report or clinical document associated with a study (if available for that study).
	Reset - Remove all annotations and manipulations from the active viewport.
	MIP/MPR - Hover over this icon to construct standard MPR, MIP, or volume rendering (VR) images for a study.
	Logout - Log out of ZFP.
	<p>When this icon is selected, lossless progression is disabled for image display. Images are loaded as lossy only. When this icon is not selected, lossless progression is enabled for image display. This is the default.</p> <p>Note: The tooltip shows the current network bandwidth. ZFP tests your network speed after downloading all lossy images for a study. If your network speed is 25 Mbps or lower, then ZFP selects this icon while the images are loading so that you can view images faster.</p>
	About - Open the About dialog box. This icon is for the ZFP desktop version only.
	<p>Help - Open a drop-down menu:</p> <ul style="list-style-type: none"> • User Manual - Open the ZFP online help. • System Prerequisites - Display the System Prerequisites dialog box. This dialog box contains the optimal ZFP system prerequisites related to browsers, operating systems, display size, and product model. • Help Guide - Display static ZFP UI images with tips that highlight basic areas of functionality, such as how to navigate the Study Search screen, and tips for image navigation and manipulation. Not supported on mobile phones. • Security and Privacy - Display the Security and Privacy screen. This informational screen is intended to remind users about their privacy and security obligations

General Electric reserves the right to make changes in specifications and features, or discontinue the product or service described at any time, without notice or obligation. This does not constitute a representation or warranty or documentation regarding the product featured. All illustrations or examples are provided for informational or reference purposes and/or as fictional examples only. Your product features and configuration may be different than those shown. Information contained herein is proprietary to GE. No part of this publication may be reproduced for any purpose without written permission of GE.



GE Healthcare, 500 W. Monroe Street, Chicago, IL 60661 USA